Multifunctional stretcher chair



SEATING

Day Hospital Emergency room Patient transport Multifunctional Triage

Frequent use

Transportation in SECURITY Inpatient and outpatient reception





Standard equipment

Effortless, safe transportation with

ergonomic handle grips

The stretcher changes its appearance

ATHENA 2.0 is a combination of passion, elegance and a lot of experience.

The study for the details and the technological research are evident, however the real difference is provided by its practical functional solutions. Suitable to accommodate patient's of every size, it ensures positive feeling and incomparable relaxation. It helps to perform all therapeutic treatments in a perfect postural conditions.

The advantageous comfort of an armchair, as comfortable as a bed and as functional as a stretcher. Suitable for any clinical situation, it favours unique postural conditions allowing to accomodate patients of different sizes. The project represents an innovative furnishing solution, reduces the patient's psychological discomfort without sacrificing the performance of the classic stretcher.





Other features of the medical device

- Painted steel frame in thermosetting powder.
- Foldable side rails couple Equipment code AC132.
- No. 4 twin 150 mm wheels Equipment code AC5P/1.
- Simultaneous wheel lock via bilateral pedal and with directional system Equipment code AC7P2G.
- Bilateral front/rear pedal brake Equipment code AC7P3.
- Electrically driven movements through handset:
 - Motorized variable height 300 mm Equipment code AC28/200.
 - Backrest section
 - Leg Equipment code AC12.
 - Trendelenburg.
 - 0°/-15° motorized seat Equipment code AC35/1.
- Legrest Equipment code AC13/SH1.
- Oxygen tank holder Equipment code A95P.
- Double handle with ergonomic grip Equipment code AC117.
- 13 buttons handset to achieve all the available movement, it features an inhibition key and pre-setted position (equipment code AC6MP). Chair's movements are controlled by a 10 buttons handset with a 3,5mtl coiled cable; For patient's safety purpose the handset is provided with an inhibition key, Equipment code AC34P. The handset is user friendly and has visual pictograms useful for partially sighted people. Insulation IP66.
- Bilateral handset support Equipment code AC25.
- Powered by interchangeable 4.5 Ah battery (no. 2 supplied) with desktop charger Supply code AC17P/A.
- Supplementary headrest pillow Equipment code AC27.
- Seat/leg section cushions easily removable.
- Soft and removable coating for the footrest Equipment code AC16/1.
- ABS cover for base equipment code AC83.
- Cushions made in soft foam height 100 mm.
- Upholstery: with anti-bacterial properties, flame retardant class 11M, antimicotic, ecologic, latex free, phtalate free. Resistant to: alchool, sodium hypochlorite max 5%, a wide range of common used detrgent, liquid and other biological fluids. Equipment code AC87/1.








Optional Components

AC109



AC109/1

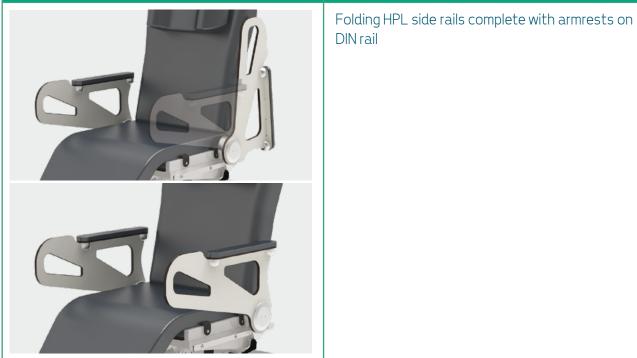




5

Optional Components


AC133

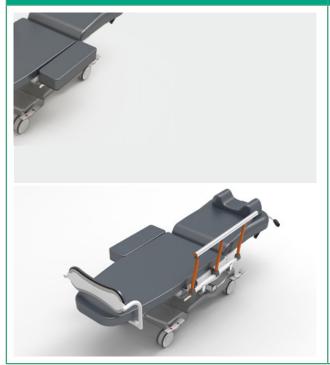


Seat width extendable by 20 cm on DIN rail

Fifth directional wheel with foot pedal activation

AC118

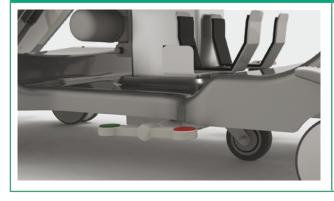

AC129

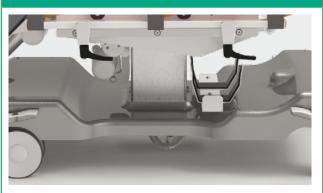

 ${\it Self-leveling\ shock-absorbing\ fifth\ swivel\ wheel}$





Optional Components

AC144



Multifunction armrest on DIN rail

AC4P/3



DIN rail-mounted steel pipe support rod

AC145



DIN clamp, universal for accessory housing



Do you want to implement the device? Frame the QR code and discover all the optional components available by browsing the catalog.

Seating


Data sheet

CND/EMDN Code	V08020101
GMDN Code	38447
No. progressive/R	2433997/R
UDI-DI Basic	805771740DITRGR
Product Code	GB0228.SP Painted steel
Intended Use	Multifunctional chair
Manufacturer	GARDHEN BILANCE SRL
Total weight	80 kg
Wheels	N°4 twin wheels 150 mm with braking system
Power supply	100 - 240 Vac 50/60 Hz
Electric motors	Low voltage (24 V)
Max absorption	4A
Insulation	IP44
Maximum Scope	200 kg
Safe Workload	235 kg

Main Cord





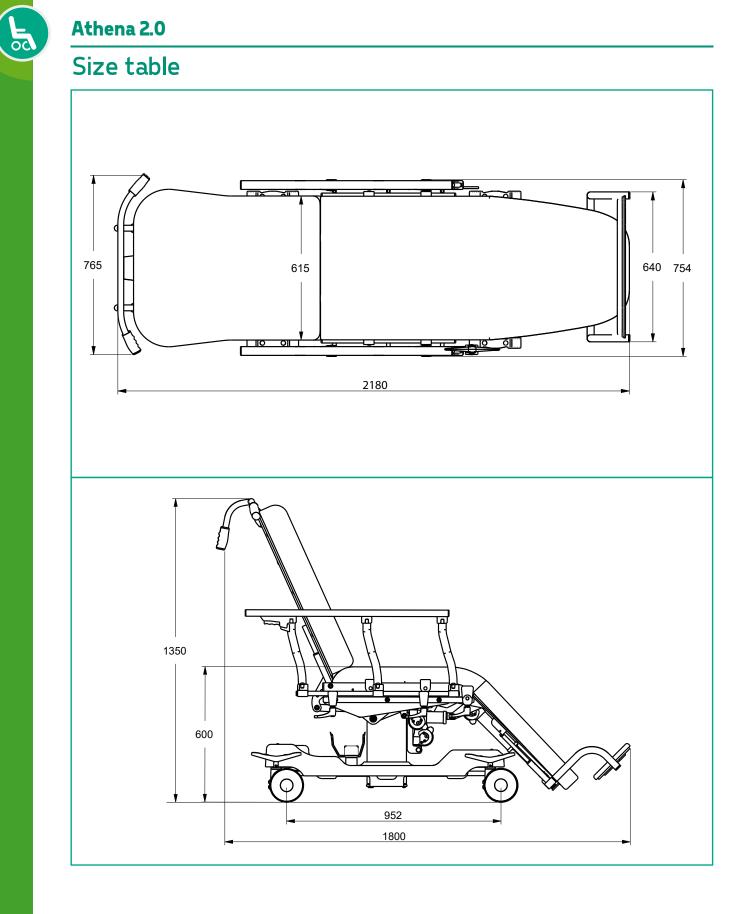

www.gardhenbilance.it



Images reproduced in this sheet are purely indicative and dimensional values are nominal. It is the right of the manufacturer to make changes to the designs without any notice. For identification of standard equipment and for the choice of any additional components refer to the catalog optional components and/or commercial proposals. In case of discrepant data, the content of the commercial proposals shall prevail Dep. EN_ 19 rev. 01/07/2024

Gardhen Bilance Srl

Via G. Luraghi c/o Consorzio II Sole - lotto S -80038 Pomigliano d'Arco (NA) - Italy Tel. +39.081.8692160 pbx - Fax +39.081.8692460 E-mail: info@gardhenbilance.it PEC: gardhenbilance@pec.it