

Stephen H Eco

Multifunctional chair



SEATING

Frequent Usage

Chemotherapy
Infusions Therapy
Day Hospital
Blood donor
Blood samples
Emergency
Dialysis
Patient transport
Dermatology
Cosmetics
Multifunctional

VERSION 1
BACKREST
STEPHEN H ANATOMICAL

VERSION 2
BACKREST
STEPHEN H FLAT

High
COMFORT

Inpatient and outpatient treatment



www.gardenbalance.it



Cushions fully removable
to facilitate the activities
of sanitising the device



**Multi-adjustable
ergonomic armrests**



**Wheels with
braking system**
activated by
bilateral levers.



Multi-adjustable armrests

Ergonomic armrests equipped with a special manual mechanism with multiple adjustment for positioning in several directions. In addition, the support placed bilaterally on the armrests (equipment code AC25P) allows the hand control to be conveniently stored when not in use.



Solidal to the Backrest



Rotating



Tiltable



Overtured at 90°



Movements

Synchronised back and leg sections

The movements of the synchronised back and leg sections are controlled by dedicated buttons on the handset.



Activated by
Handset



Movements simultaneous
of the back and leg sections






Movements

Trendelenburg

The Trendelenburg position can be obtained using the dedicated buttons on the handset.

Activated by
Handset

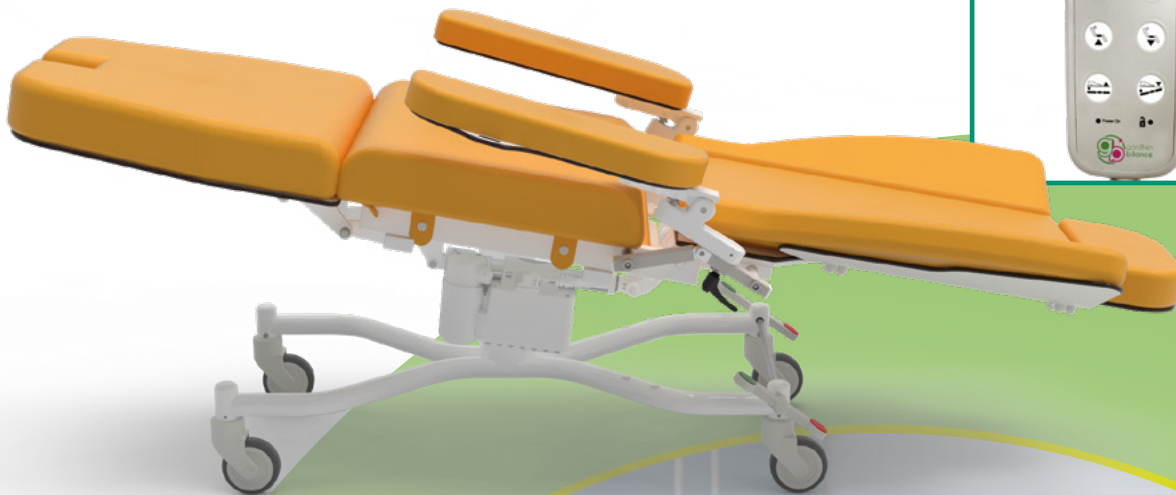

Recliner

Another example of movement





Other features of the medical device

- Painted steel frame in thermosetting powder
- Lying surface divided in 4 sections (headrest, backrest, seat and leg).
- Simultaneous movements of the back and leg sections.
- Trendelenburg position (alternative shock position).
- Headrest joint to the backrest.
- Armrest's movements synchronized with the backrest. The armrests are completely removable and can be rotate, tilted and overturned at 90°; (please refer to the accessories catalogue for other version of armrests).
- No. 4 wheels ø 100 mm antistatic, sturdy, stainless and swivable.
- Simultaneous brake on rear wheels in chromed steel - Equipment code AC7P1G.
- Chair's movements are controlled by handset with a 3,5 mtl coiled cable. The handset is user friendly and has visual pictograms useful for partially sighted people. Handset is equipped by 10 buttons with 2 memories.
For patient's safety purpose the handset is provided with an inhibition key - Equipment code AC34P. Insulation IP66.
- Bilateral handset support - Equipment code AC25P.
- Easily removable soft foam independent cushions.
- Upholstery in flame retardant leatherette UNI 9175/87 e 9175FA-1/94 class 11M - Equipment code AC87/1.

Do you want to change the colour of the upholstery and frame?

Frame or click the QR codes and discover how to personalise your device by consulting our colour cards*.

*If no preference is expressed, the device will be delivered in the available colour.

Upholstery
Leatherette

Coating
Silicon

Frame





Do you want to improve the device?

Scan the QR Code to find out all the available accessories listed on our catalogue.





Dimensions

Standard version

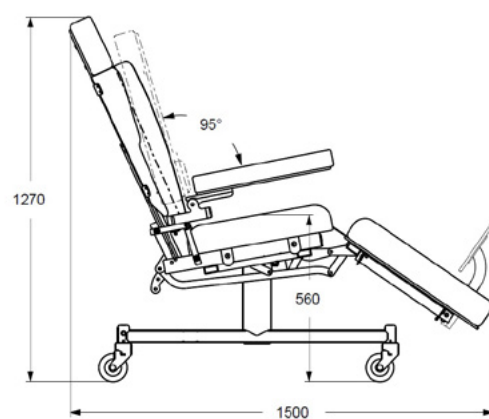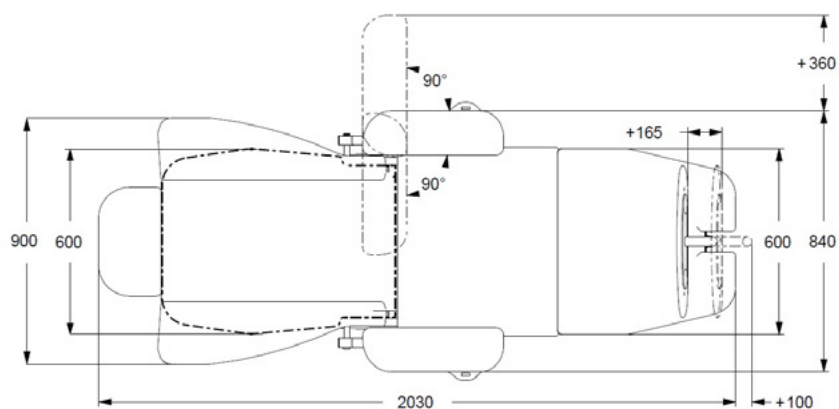

Junior version

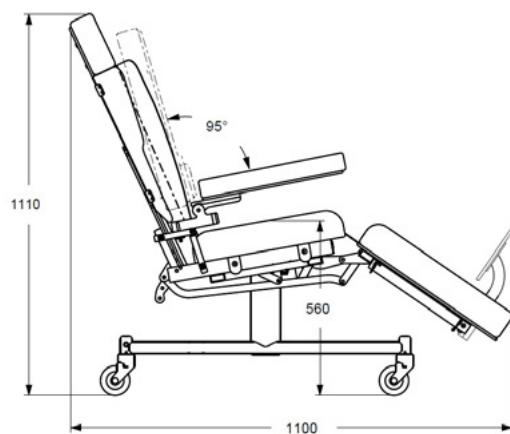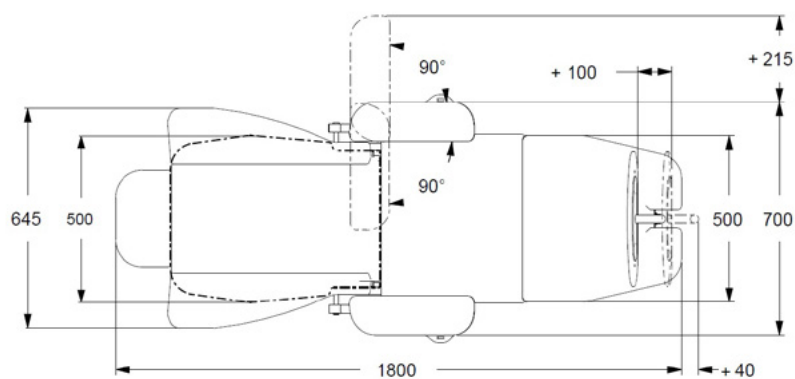



Technical sheet

CND/EMDN code	V08020101
GMDN Code	38447
N. progressive/R	2433953/R
UDI-DI di Base	805771740POLPE
ID Product	GB0005.JP-1 Painted steel
Version 1	GB0005.JS-1 Stainless
Backrest	GB0005.SP-1 Painted steel
Stephen H Anatomical	GB0005.SS-1 Stainless
ID Product	GB0005.JP-2 APainted steel
Version 2	GB0005.JS-2 Stainless
Backrest	GB0005.SP-2 Painted steel
Stephen H Flat	GB0005.SS-2 Stainless
Inteded use	Multifunctional chair
Manufacturer	GARDHEN BILANCE SRL
Overall weight	Standard 70 kg / Junior 50 kg
Wheels	No.4 ø 100 mm swivable and stainless with braking system
Power	100 - 240 Vac 50/60 Hz
Electric motors	Low voltage (24 V)
Max electrical input	4A
Insulation	IP44
Max load	Standard 200 kg / Junior 150 kg
Safety working load	Standard 235 kg / Junior 185 kg

Main cord



Type F
Schuko



Type L
16A



Type I
Australia



Type B
Canada and USA



Type G
UK plug

www.gardhenbalance.it



Gardhen Balance S.r.l.

Via G. Luraghi c/o Consorzio Il Sole - lotto S -
80038 Pomigliano d'Arco (NA) - Italy
Tel. +39.081.8692160 pbx - Fax +39.081.8692460
E-mail: info@gardhenbalance.it
PEC: gardhenbalance@pec.it

Pictures contained in this brochure is for illustrative purpose only. The manufacturer reserves the right to modify the projects without notice. For accessories identification please refer to the accessory catalogue or commercial offer. When data are different please refer to commercial offer.

Dep. EN_69 rev. 17/10/2024