# Stephen H Comfort 4.0 Multifunctional chair



**SEATING** 

Frequent Usage

Chemotherapy

Infusion therapy

Day Hospital

**Blood Donation** 

**Blood Sample** 

**Emergency Room** 

Dialysis

Patient transport

Dermatology/ Cosmetics

Multifunctional

version with motorized column

HightCOMFORT Inpatient and outpatient therapy



www.gardhenbilance.it



# Multi-adjustable armrests

Armrests ergonomic armrests equipped with special manual multiple adjustment mechanism for positioning in multiple directions. In addition, the support placed bilaterally on the armrests (equipment code AC25P) allows convenient storage of the hand control when not in use.



Synchronized with the backrest



**Swivable** 



Height adjustable



Removable



Tiltable









#### **Movements**

### Independent back and leg sections

Independent movements of the back and leg sections are controlled by dedicated buttons on the hand control.



# **Movements** Trendelenburg

The Trendelenburg position can be obtained at the dedicated button on the hand control.







#### Other features of the medical device

- Thermoset powder coated steel frame.
- Top divided into 4 sections: headrest, backrest, seat and legs.
- Simultaneous and independent movement of backrest and leg sections by 2 motors equipment code AC12.
- Trendelenburg position.
- Armrests integral to the backrest movement, with swivel function, tiltable up/down, 90° tilt and height adjustment; (see accessories catalog for other armrest type).
- No. 4 durable, antistatic, stainless, swivel wheels Ø 100 mm.
- Extremely stable headrest, removable and height adjustable to 5 positions.
- Simultaneous brake galvanized steel rear wheels Equipment code AC7P1G.
- All chair movements are controlled by a perimeter hand control equipped with a 3.5 m coiled cable. The hand control is easy to use as it is made with large raised pictograms and is also suitable for visually impaired patients. The hand control is equipped with 10 keys with 2 memories. For patient safety, the hand control is equipped with a key for control inhibition Equipment code AC34P. IP66 insulation.
- Double-sided push-button panel support Equipment code AC25P.
- ABS cover for base Equipment code AC83.
- ABS seat cover Equipment code AC84C.
- ABS cover for leg rest Equipment code AC85C.
- ABS backrest cover Equipment code AC86C.
- Cushions made of soft foam.
- Fire-retardant leatherette upholstery UNI 9175/87 and 9175FA-1/94 class 1IM Equipment code AC87/1.

# Do you want to change the color of the upholstery and frame? Frame or click the QR codes and find out how to customize your device by browsing our color cards\*. \*If no preference is expressed, the device will be supplied with the available color. \*Upholstery Leatherette \*Upholstery Silicon \*Frame





#### Do you want to improve the device?

Frame the QR code and discover all the accessories available by browsing the catalog.



#### **Technical Sheet**

 CND/EMDN Code
 V08020101

 GMDN Code
 38447

No. progressive/R 2234628/R

UDI-DI di Base **805771740POLPE** 

ID Product GB0004.BP-4 Painted steel

GB0004.SP-4 Painted steel

Intended Use Multifunctional chair
Manufacturer GARDHEN BILANCE SRL

Total weight Standard 70 kg / Bariatric 100 kg
Wheels No. 4 stainless swiveling 100 mm

with braking system

Power 100 - 240 Vac 50/60 Hz Electric motors Low voltage (24 V)

Max absorption 4A Insulation IP44

Max load capacity Standard 200 kg / Bariatric 300 kg
Safety working load Standard 235 kg / Bariatric 335 kg

## Main cord



Type F Schuko



Type L 16A



Type I Australia




Type B Canada and USA



Type G UK plug



# **Dimensions**

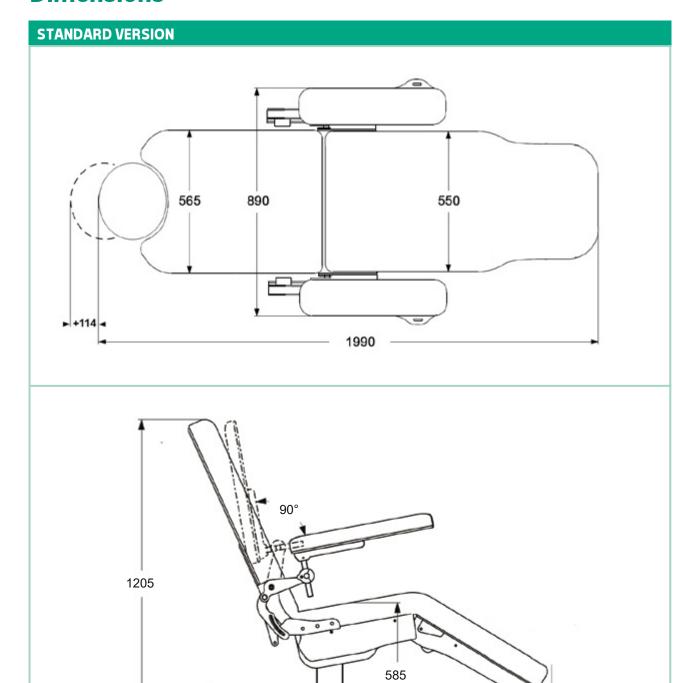


- 1480 -





#### **Dimensions**

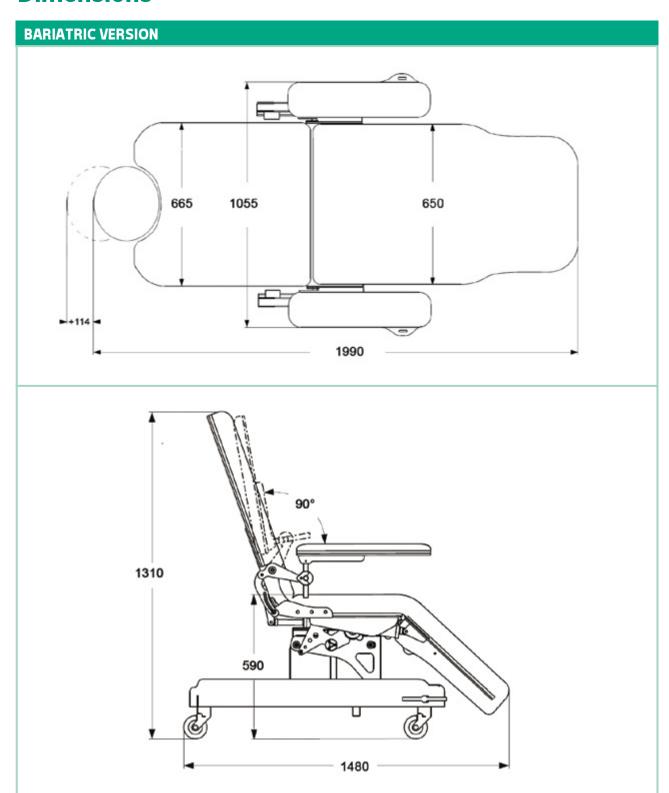

#### www.gardhenbilance.it











Via G. Luraghi c/o Consorzio II Sole - lotto S -80038 Pomigliano d'Arco (NA) - Italy Tel. +39.081.8692160 pbx - Fax +39.081.8692460 E-mail: info@gardhenbilance.it PEC: gardhenbilance@pec.it

Images reproduced in this sheet are purely indicative and dimensional values are nominal. It is the manufacturer's right to make changes to designs without prior notice. Refer to the accessories catalog and/or commercial proposals for identification of standard equipment and selection of any additional components. In case of discrepant data, the content of the commercial proposals shall prevail. Dep. EN\_149 rev.29/04/2024